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
Handwritten signatures and stamps on the right side of the document.

**TRU TRAC TRACTION UNIT AND TRACTION TABLES** 

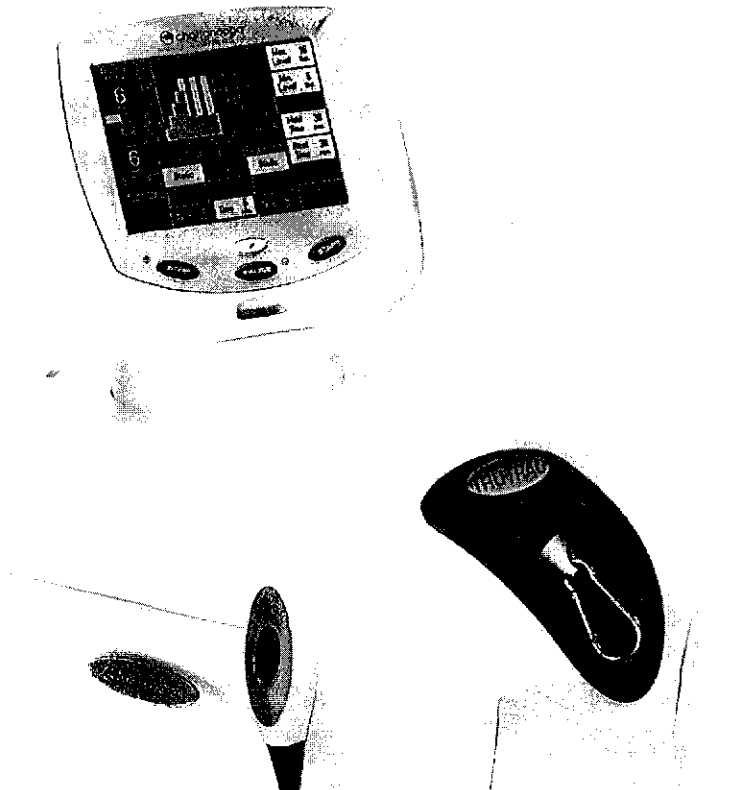**FEATURES**

**Tru-Trac Traction Unit**

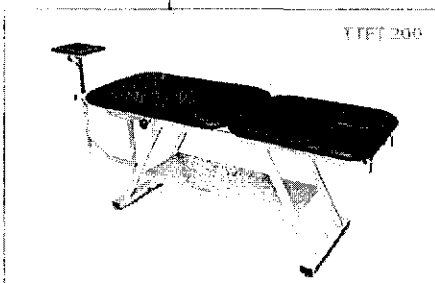
- Static, intermittent, and cycling Traction 4.1, 4.1.1, 4.1.2, 4.1.3
- User-defined hold, rest and treatment times
- 10 User-defined protocol slots to store customized settings 4.10
- Digital monochromatic touch screen interface that pivots 270° 4.7, 4.8, 4.9
- Easily store treatment parameters, pain scales, and pain maps on Patient Data Cards 4.12

**TTEF-200**

The TTEF-200 is a two-section, 71 cm wide table with an adjustable height from 48 cm - 96 cm and a working load of 181 kg. Optional retractable casters are available. Traction unit and traction stool sold separately. Available in all upholstery colors shown in chart.

**TTFF-200**

The TTFF-200 is a two-section, 71 cm wide table with a fixed height 81 cm from top to floor and a working load of up to 159 kg. Traction unit and traction stool sold separately. Available in all upholstery colors shown in chart.

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**TECHNICAL SPECIFICATIONS**

Tru-Trac Traction Device	
Mains Power:	100 - 240V, 50/60 Hz 3.1
Power Consumption:	75 W
Current Consumption:	3.2 Amps Max
Weight:	14 kg (30 lbs)
Dimensions*:	45 x 24 x 45 cm
Electrical Safety Class:	Class 1, Type B
Safety Tests:	IEC/EN 60601-1, IEC/EN 60601-1-2, CAN C22.2 No. 601.1-M90 w/A2 - Meets MDD 93/42/EEC, CE 0413 3.2

**Economy Hi-Lo Traction Table**

Mains Power:	230V, 50/60 Hz; 120V, 60 Hz
Current:	230V - 1.0 A; 120V - 1.6 A
Power Input:	230V - 230 W; 120V - 200 W
Weight:	64 kg (140 lb)
Dimensions*:	196 x 71 x 48 to 96 cm
Electrical Safety Class:	Class 1, Type B

\* Dimensions are listed as L x W x H

**COLOR OPTIONS**

**Standard Colors**

Black (002)	Imperial Blue (001)
Burgundy (004)	Dove (007)
Midnight Gray (033)	Stone Blue (061)

NOTICE: Color charts are produced within the limitations of lithographic color printing and are intended as a general guide only.

# SPECIFICATIONS

Iru-Trac® Traction Unit

## UNIT DIMENSIONS AND SPECIFICATIONS

Width	24 cm (9.5 in)
Depth	45 cm (17.5 in)
Height	45 cm (17.5 in)
<b>Weight</b>	
Standard Weight	14 kg (30 lbs/134 N)
Shipping Weight	18 kg (40 lbs/178 N)
<b>Power</b>	
Voltage	100V-240V (50/60Hz)
Power Consumption	75 VA
Current Consumption	3.2 Amps Max
Electrical Class	Class I
Electrical Type	Type B 

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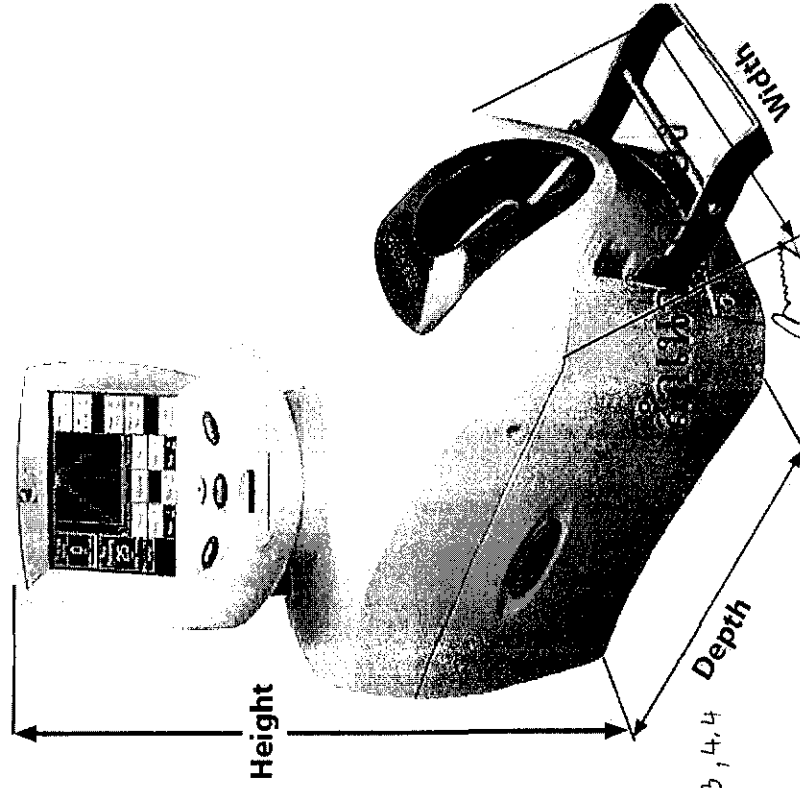
ufam CLINIQUE CO.,LTD.

## Traction Modes

Static, Intermittent, Cyclic, and their combinations.

Traction Type: Mechanical

LCD Display: High contrast monochrome touch screen

## Traction Tension

Parameters	Minimum	Maximum	Increments
Traction Period	1 minute	99 minutes	1 minute 4, 6
Hold Period	0 seconds	99 seconds	1 second 4, 5
Rest Period	0 seconds	99 seconds	1 second
Traction Tension	0 kg (0 lb/0 N)	90 kg (200 lb/890 N)	1 kg (1 lb/1 N) 4, 3, 4, 4
Progressive and Regressive Steps	1 step	9 steps	1 step 4-2

# OPERATION

Tru-Trac® Traction Unit

## OPERATING CONTROLS - HOME SCREEN

### (11) Minimum Level

Displays the minimum traction tension entered during setup. To enter the minimum traction tension, press the Min Level button, then enter the desired minimum setting, using the number keypad. Press the Return Arrow button to accept.

**NOTE:** The Minimum Traction Tension can be changed during the traction therapy session by pressing the Min Level button, then use the Up or Down Arrow buttons to make changes in 1 kg (1 lb or 1 N) increments. Press the Return Arrow button to accept.

### (12) Maximum Level

Displays the maximum traction tension entered during setup. To enter the maximum traction tension, press the Max Level button, then enter the desired maximum setting, using the number keypad. Press the Return Arrow button to accept.

If the Maximum Level of traction tension is greater than 17 kg (39 lbs or 166 N), a message will be displayed saying "Cervical traction should not be performed with a tension greater than 17 kg (39 lbs or 166 N); indicate the type of traction to be performed". The Lumbar Button must be pressed to start treatment, otherwise press the Cervical button and ~~then enter the Maximum Level of traction tension to be lower than 18 kg (40 lbs or 176 N).~~

**NOTE:** The Maximum Traction Tension can be changed during the traction therapy session by pressing the Max Level button, then use the Up or Down Arrow buttons to make changes in 1 kg (1 lb or 1 N) increments. Press the Return Arrow button to accept.

### (13) Clinical Resources

This button offers the following features of the traction unit: (See Patient Data Card and User Protocols in the Operation Section for more information)

#### • Patient Card

- Patient Card (Patient Name)
- Edit Treatment Being Setup
- Erase Patient Card

#### • Protocols

- Retrieve Protocol
- Save Protocol

#### • Utilities

- Unit Settings (see Pages 17-18 for more information)

### (14) Stop 4.7

Stops the treatment program. Traction tension goes to zero.

### (15) Pause 4.7

Pauses the treatment program. Traction tension goes to zero. Press the Pause Button again to resume treatment program and traction tension.

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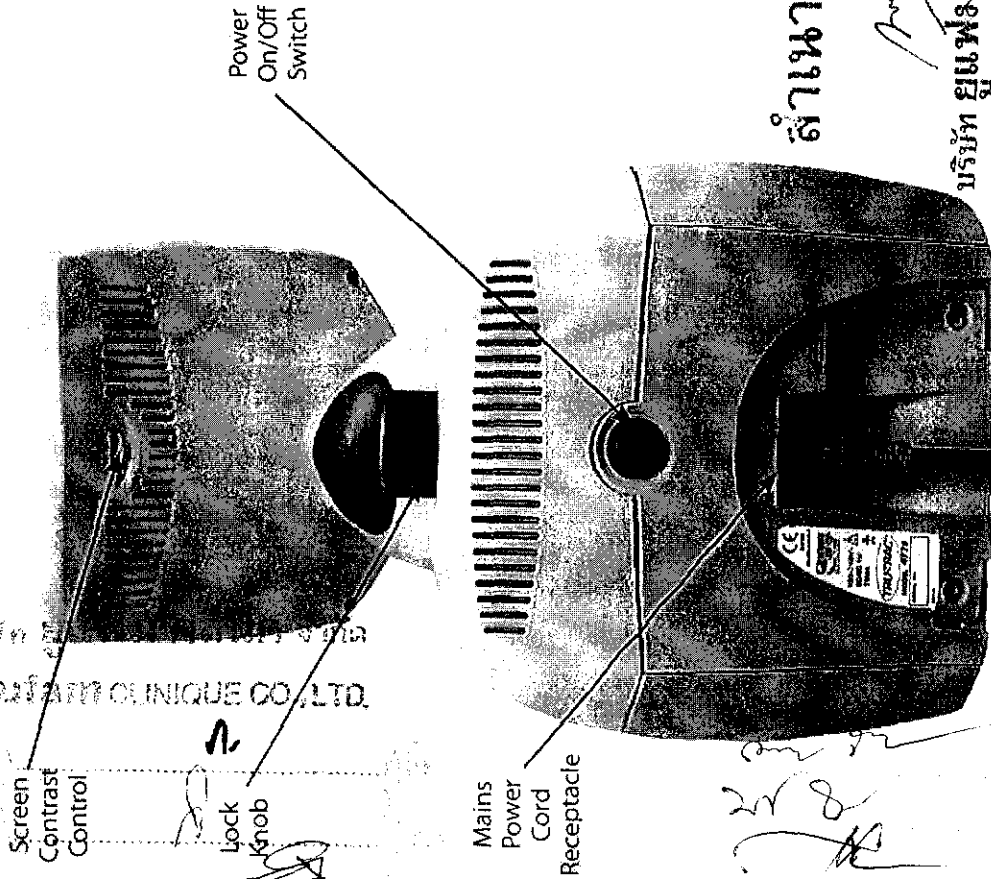
Tru-Trac® Traction Unit


## BASE UNIT, REAR VIEW

The rear view of the base unit is the site of electric current control and surge protection, and serves as a connection point for both the traction controls and safety features of the Tru-Trac traction unit.


### Power On/Off Switch

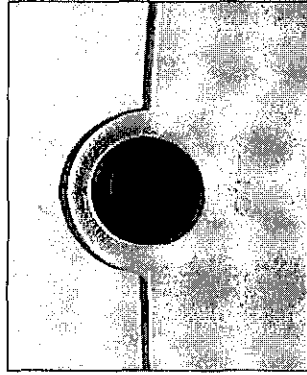The Power On/Off Switch is a toggle switch located on the back of the unit. This switch controls the flow of electricity from the outlet to the unit.

### Mains Power Cord Receptacle

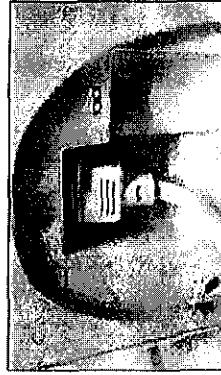The Mains Power Cord Receptacle accepts the Mains Power Cord (female end).


TOC

### Screen Contrast Control 4,9

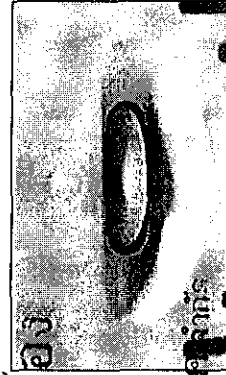The Screen Contrast Control is used to set a comfortable viewing of brightness of the touch screen user interface.

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
# OPERATION
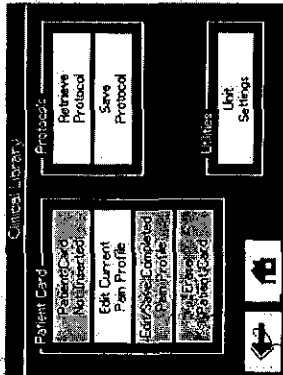
## USER PROTOCOLS

This library is a series of protocols created by the user and stored in the unit memory. The following information gives general instructions as to setup, saving, and access of User Protocols. Should the Restore Default Protocol Button be pressed, through the Utilities section of the Clinical Resources screen, all User Protocols will be permanently removed from the unit.

**NOTE:** Ten User Protocols can be saved as well as a Default Protocol. 4.11


### Set Up and Saving User Protocols

Select and edit the parameters of the desired treatment from the Home Screen.

Press the Clinical Resources Button on the front of the traction unit.

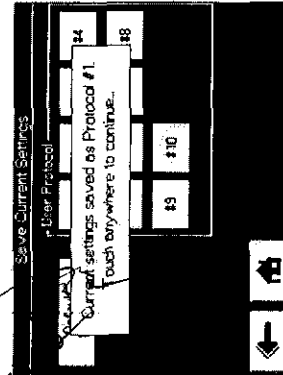
Press the Save Protocol Button located in the Protocols section of the menu.

Select the number of User Protocol to save the desired treatment to or select the Default Protocol Button to save treatment as the default.

A message will be displayed saying **Current Settings saved as Protocol #.** Touch the screen to continue.


### Accessing User Protocols

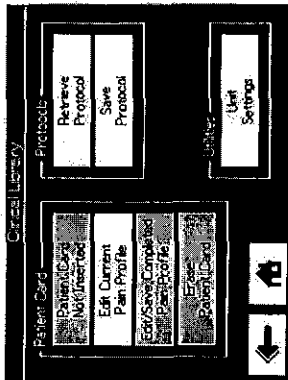Press the Clinical Resources Button on the front of the traction unit.

Press the Retrieve Protocol Button located in the Protocols section of the menu.

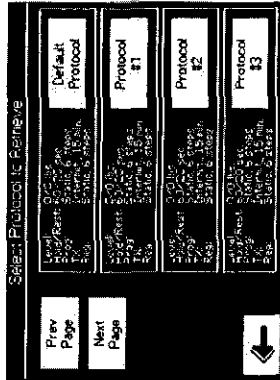The list of the User Protocols is located on the right side and the Default Protocol is located on the left side. Press the Page Up or Page Down Buttons to review the User Protocols.

Select the desired User Protocol by pressing the number of the User Protocol button.

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ufam CLINIQUE CO., LTD.